

Trainingvideo-upload preparation

Prepare an empty USB stick.

Click on the download-link with the newest video package for your device model and save the file on your computer.

Extract this file directly into the main directory of your USB stick, using the software tools [7Zip 32Bit](#) or [7Zip 64Bit](#).

Short instruction on how to create the Upload USB stick:

1. Insert an USB stick to the switched on device, it will be mounted to a volume letter like F:
2. Extract this file directly into the main directory of your USB stick using 7zip. You will get a file ending with .sh and a folder with the number of your device model (12 or 16).
3. Insert this USB stick into the switched on Plusoptix. Follow the instructions on the screen until the upload is confirmed.
4. Remove the USB stick. The videos are now available on your device.

