

## Binocular Autorefractor



reddot award 2014  
winner



### Short Manual 9

### “Downloading software updates”

Plusoptix GmbH  
Neumeyerstrasse 48  
90411 Nuremberg  
Germany  
[www.plusoptix.com](http://www.plusoptix.com)

## Table of Contents

1	Downloading software updates – an overview .....	3
2	Downloading software updates on the workplace computer .....	3
3	Downloading the software update directly onto the device (LAN / WLAN or mini USB) ...	4
4	Downloading Software Update via LAN / WLAN (only A12C and A16) .....	5
5	Downloading Software Update via mini USB-cable (only A12C and A12R) .....	6

## List of figures

Figure 1: Downloading software updates.....	4
Figure 2: Downloading the software update via LAN / WLAN .....	5
Figure 3: Software Update via mini USB cable .....	6



This manual is a supplementary short manual. For general information about your Plusoptix device, such as its intended use, basic handling, technical information, maintenance, service, and warranty information, please refer to the user manual that was delivered with your device. This user manual can also be downloaded on our homepage [plusoptix.com](http://plusoptix.com), under support.

## List of further supplementary short manuals

1. Checking the scope of delivery and learning about the device
2. Adjusting settings and setting up WLAN
3. Preparing and performing a measurement
4. Viewing the measurement results
5. Entering, retrieving or deleting patient data
6. Documenting measurement results electronically
7. Documenting measurement results on paper
8. Exporting back-ups and reports (only plusoptiX A12C and A16)
9. Downloading software updates
10. Troubleshooting malfunctions and measurement interruptions

## 1 Downloading software updates – an overview

We recommend checking regularly to see if there is a new software version to download. If your device is already up-to-date, the message “The software version is up-to-date” will be shown. To download a software update, you will need an empty SD card (plusoptiX A12C / A12R) or USB stick (plusoptiX A16).

You have following options to download the latest software version:

- Downloading software updates on the workplace computer (see chapter 2)
- Downloading the software update directly onto the device (LAN / WLAN or mini USB) (see chapter 3)
- Downloading Software Update via LAN / WLAN (only A12C and A16) (see chapter 4)
- Downloading Software Update via mini USB-cable (only A12C and A12R) (see chapter 5)



**Attention:**

During the update process, all the data saved on the SD card or USB stick will be deleted. For this reason, make sure you only use an empty storage medium.



**Note:**

After the software update has been installed, you will see a message about formatting the SD card or USB stick. Confirm this message if you want to use the SD card or USB stick to save Measurement reports after the update. Do not format it if you want to use it to update further Plusoptix devices.

## 2 Downloading software updates on the workplace computer

If you cannot establish an internet connection to the device, the software update can also be downloaded to the workplace computer.

The latest software update and instructions to update the software can be found on our homepage ([plusoptix.com](http://plusoptix.com)), under support.

## 3 Downloading the software update directly onto the device (LAN / WLAN or mini USB)

If you want to download the software update using LAN/WLAN or Mini-USB, take the following precautions for updating the software:

- Insert an empty SD card (plusoptiX A12C / A12R) or USB stick (plusoptiX A16)
- Connect the Plusoptix device to the power network
- Establish internet access (either over a LAN / WLAN-connection or over a mini USB cable to your workplace computer)

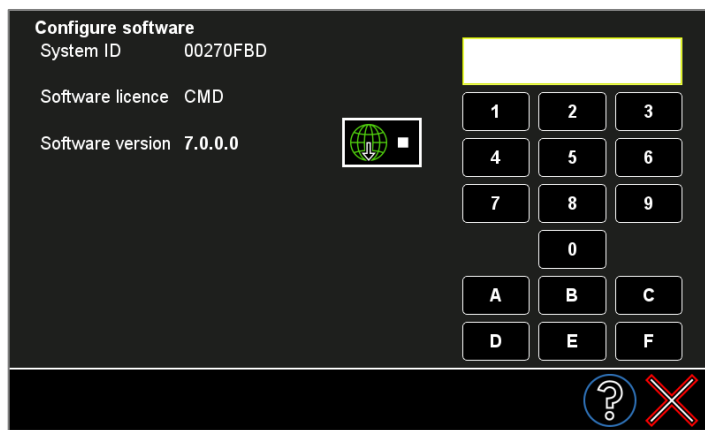


Figure 1: Downloading software updates

- Touch the button to configure the software (1) in the settings.
- Then touch the button (2) to download the latest software
- Confirm with the green checkmark (3). You will be guided through the download (see figure 2).

When downloading, the software update will be saved on the SD card or USB stick and a confirmation will be displayed. As soon as you touch OK, the device will automatically restart and install the software update saved on the SD card or USB stick.



## 4 Downloading Software Update via LAN / WLAN (only A12C and A16)

With active WLAN connection in the plusoptix A12C model or active LAN or WLAN connection in the plusoptix A16 model, you will be guided through the update with the help of the following status messages (see Figure 2).



**Note:**

Depending on the device, you either need an empty SD card (plusoptix A12C / A12R) or an empty USB stick (plusoptix A16). The following status messages are the same for the Wplusoptix A12CW and “plusoptix A16” model, and only differ in the storage medium.

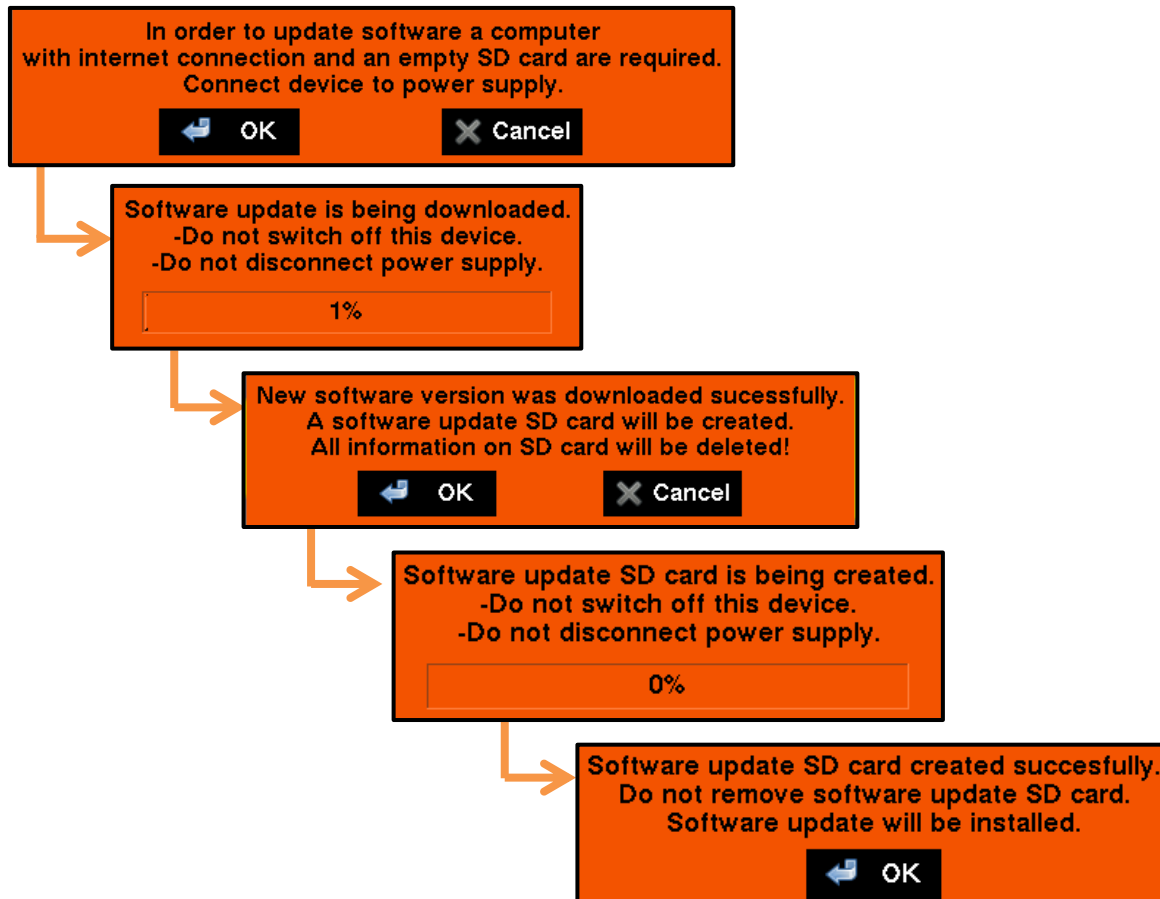


Figure 2: Downloading the software update via LAN / WLAN

## 5 Downloading Software Update via mini USB-cable (only A12C and A12R)

If there is no WLAN-connection available, the software update can be downloaded via mini USB cable (see Figure 3).



**Note:**

Depending on the internet connection, a software update via mini USB cable can take about 15-25 min.

Action recommendations on the A12C / A12R

Action recommendations on your workplace

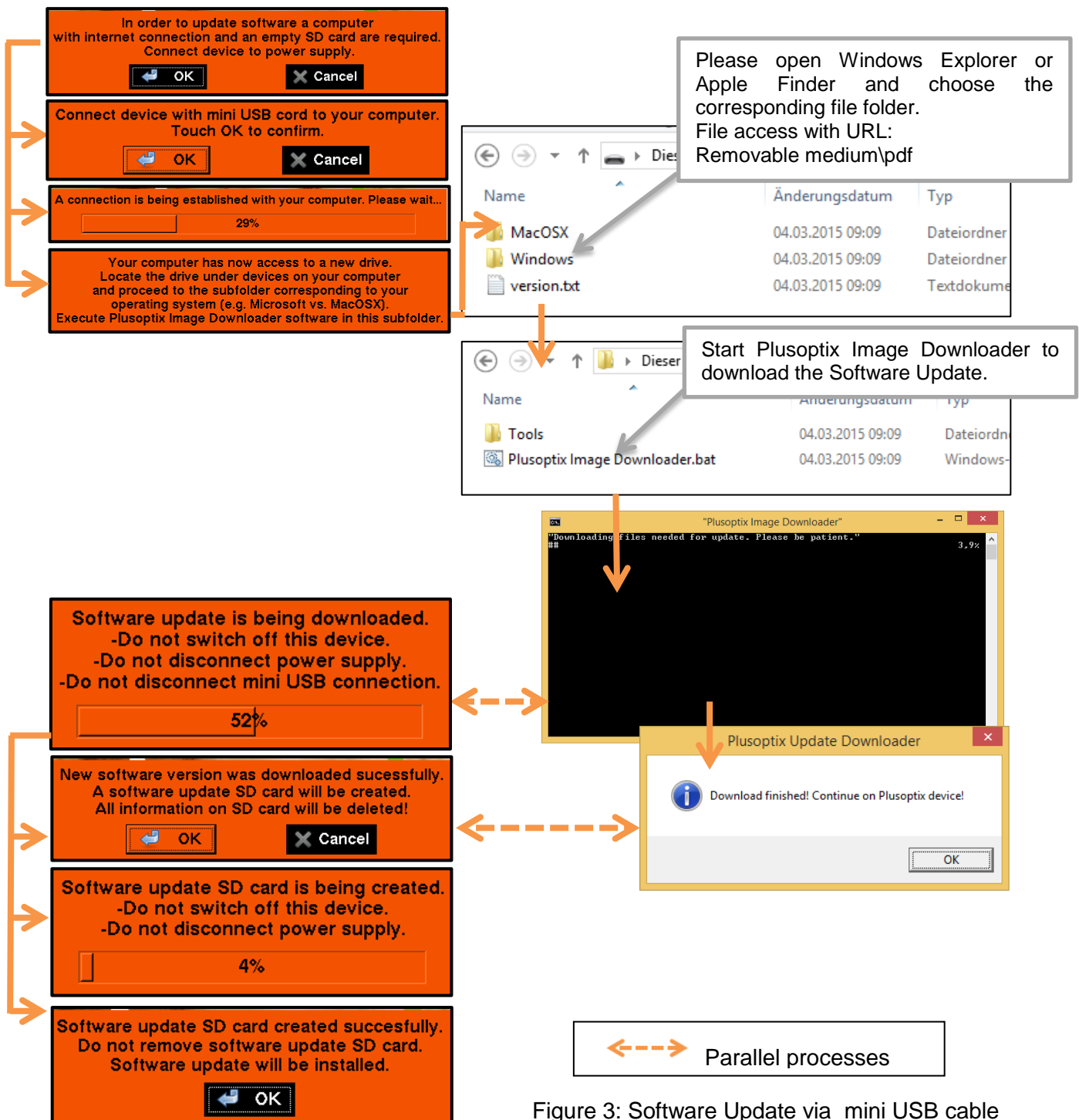


Figure 3: Software Update via mini USB cable